

## Royal Blue Tube Guide: Which Royal Blue Tube Should I Use?

### Before you get started:

- ✓ Please review the test's collection instructions in the catalog: [corewellhealth.testcatalog.org](http://corewellhealth.testcatalog.org)
- ✓ Tube manufacturers may change, *always* check the tube label to make sure you have the correct tube and it matches the description in the lab catalog.
- ✓ If you are unsure contact the department where the test is performed.

**Tube: BD Vacutainer, Trace Element, K<sub>2</sub> EDTA, 6.0 mL  
6.0 mL lavender vertical stripe on label**

**Arsenic** [LAB727]

**Cadmium** [LAB736]


**Chromium, Whole Blood** [LAB1230486]

**Cobalt, Whole Blood** [LAB1230487]

**Heavy Metals Screen** [LAB789]

**Mercury** [LAB820]

**Zinc Protoporphyrin (ZPP)** [LAB1230014]

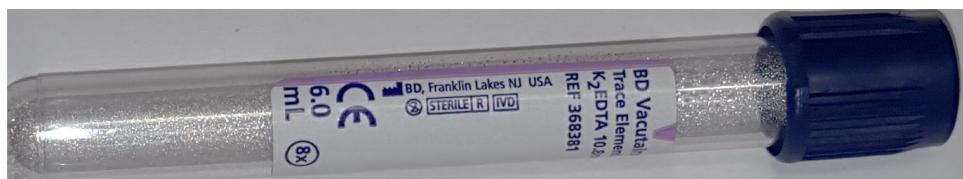

Special Order Tube – Order through Courier Supply Form

**Tube: BD Vacutainer, Trace Element, Serum, 6.0 mL  
No additive (plain) red vertical stripe on label**

**Aluminum** [LAB1231702]

**Chromium, Serum** [LAB750]

**Cobalt, Serum** [LAB3470]

**Copper (Cu)** [LAB1231751]

**Iodine** [LAB3563]

**Manganese** [LAB1230038]

**Nickel** [LAB848]

**Selenium (Se)** [LAB1231752]

**Titanium** [LAB848]

**Zinc** [LAB581]



Tube Available through Supply Chain/Workday: ITM-1066808

**Tube: BD Vacutainer, Trace Element, K<sub>2</sub> EDTA, 3.0 mL  
3.0 mL lavender vertical stripe on label**

**Lead, Blood Level** [LAB98]



Tube Available through Supply Chain/Workday: ITM-1300126